Product Model: NK-1019
Bath Lift User Manual
Manual de usuario del elevador de baño
Manuel d'utilisation de l'élévateur de bain

1. Installation and usage

- Installation and use environment confirmation

Please check the parts for damage and defects before use. Only use it if you are sure that there are no damaged or missing parts.

As shown below:

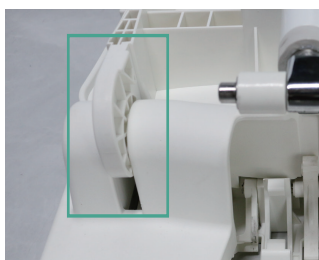
Parts included

- ① Hand control/charger
- ② Set of Wingbacks (Only helps as an aid and does not weight please **DO NOT APPLY FORCE ON WINGBACKS**)
- ③ Backrest (including linear actuator and back pad)④ Base (cushion and 6 suction cups)
- ⑤ Instruction manual


Please take out the back part of the chair and the seat part, and install the back to the seat. You can See the following figure for details.

Installation process;


① Align the backrest hook with the base guide bar.

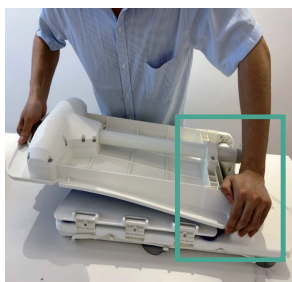
② Press the hooks on each side of the backrest to connect them to the base.

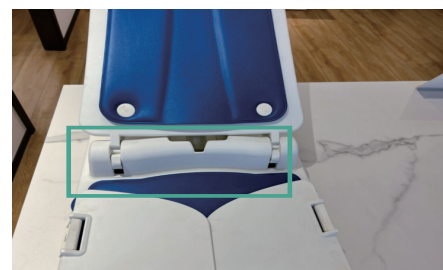③ Complete the installation of the backrest and the base.

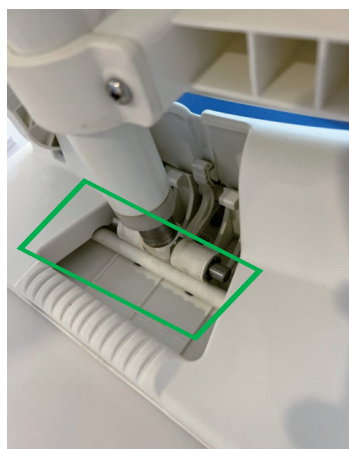④ Click on raising green sign (Pg.2 Hand Control #1) on the remote control provided in the package and make sure the top clevis of the actuator is connected


A connection is successfully made when:

- 1. Roller slowly approaches to the ground and hits the stop (Green box)
- 2. Feel a slight shake on the product

Picture attached should be the visual of the connection

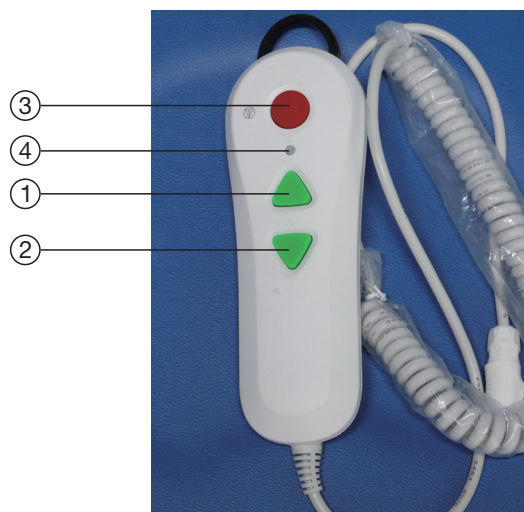
Hand control

- ① Upper triangle button: Up button - controls the rise of the bath chair
- ② Lower triangular button: Down button - controls the descent of the bath chair
- ③ Round button: Emergency stop button - control the bath chair emergency stop, and no longer descend, if you need to descend need to first press the up button and then press the down piece to descend this will **only be used when; Upper and Lower button suddenly disables and stops functioning**
- ④ Indicator light: shows the working status of the shower chair

• Start use

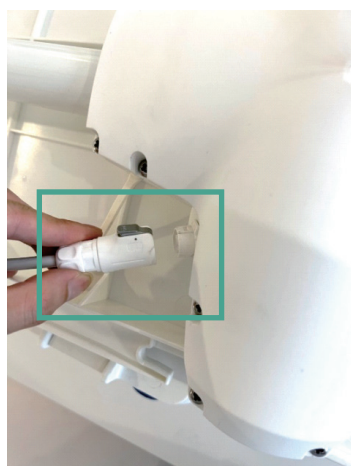- (1) First, please open the back of the bath lift, and plug the hand control with the female plug on the back.

- (2) Let the bath lift rise until the sides of the side plate lean smoothly against both sides of the bathtub;
- (3) With your back to the bathtub, sit on the outside side plate;
- (4) Maintain the above position and move the hips to the middle of the seat of the bath lift;
- (5) Put both feet into the bathtub;
- (6) Once you are in the bathtub comfortably, use the hand control down button to drop to your desired position. Keep hands and feet safe during the process;
- (7) If the down button is kept pressed, the bath lift will stop at the lowest point;
- (8) If conditions permit, have the user operate the hand control rather than the assistant. This will make the user feel more comfortable and will reduce the danger;

2. Product description

The bath lifter is designed to assist users with limited mobility to enter and leave the bathing space more easily and conveniently at home, making the bathing easier and more enjoyable. NK-1019 bath lifter fully considers the specifications of the common bathtub, so that it can be easily positioned in any standard bathtub (some non-standard or specially customized bathtubs may not be suitable). NK-1019 bath lifter is equipped with 6 suction cups at the bottom, which makes its positioning more stable and has a good anti-slip effect.

NK-1019 bath lifter is completely powered by batteries, and does not need external electrical transmission lines, so it is convenient and reliable to use; the bath lifter, except the charger, is of water-proof. You can also lift and lower the bath chair during bathing.

The NK-1019 bath lifter uses modular design, the back and the seat can be folded and disassembled, and automatically locked on the seat base when the bath lifter is raised, which can prevent the back of the chair from tilting accidentally; intelligent protection control can ensure that the hand control can lower the bath lifter when the power is insufficient.

In this user manual, you will get all the information about the NK-1019 bath lifter, including usage, storage, etc.

 **Matters needing attention**

- (1) Please use the matching charger to charge;
- (2) When charging, **keep away** from moisture, extreme temperature, debris and etc.;
- (3) The remote control is assigned with three standard colors as indicator lights. In normal use, the green light indicates that the power is sufficient, the blue light indicates that the power is insufficient, and reminds to charge. When the red light is on, it indicates that the power is seriously insufficient and needs to be charged immediately to prevent sudden power failure during the use of product.
- (5) Under normal charging conditions, the indicator light of the charger will remain red, and when the battery is fully charged, the indicator light will alter into green; the estimating charging time is about 2.5 hours.
- (6) The charger is designed for charging the hand control and **cannot** be used for other purposes;
- (7) The battery is built into the hand control, please do not disassemble the hand control without permission; when the bath chair is being transported, please confirm that the wires are in disconnected mode and are not in charging state; Prevent damage to the bath lifter and personnel caused by falling, collision, etc.
- (8) In the process of cleaning the hand control or the charger, please make sure that the plugs are disconnected; the charger can **only** be **cleaned with a dry cloth**
- (9) Load <308lb
- (10) No rush sitting

It must be charged before the first use

Lithium-ion batteries must not exceed 30% of their nominal capacity during transport, they must be charged for at least **12 hours** before first use. (Lighting on Remote control will appear green when bath lift is fully charged)

When not in use for a long time, the hand control needs to be disconnected from other parts and stored in an environment with low temperature, low humidity, no dust and no corrosive gas to maintain the best performance when the battery is used subsequently.

The following table shows the recommended storage time under the corresponding storage environment. After the storage time is exceeded, the battery should be put into use.

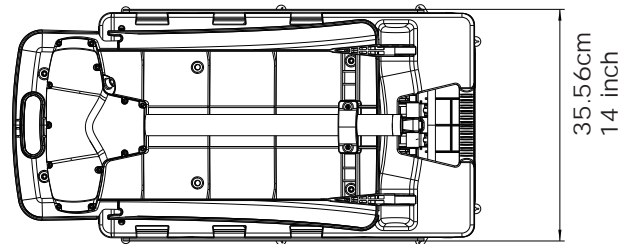
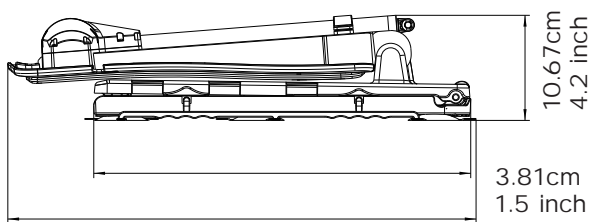
Storage time	Storage environment
1 year	10~25°C (1*), 40 ~ 70% RH (non-condensing)
3 months	0~35°C (1*), 30 ~ 75% RH (non-condensing)
1 month	-10~50°C (1*), 20 ~ 80% RH (non-condensing)

3. Appendix



Technical Specifications

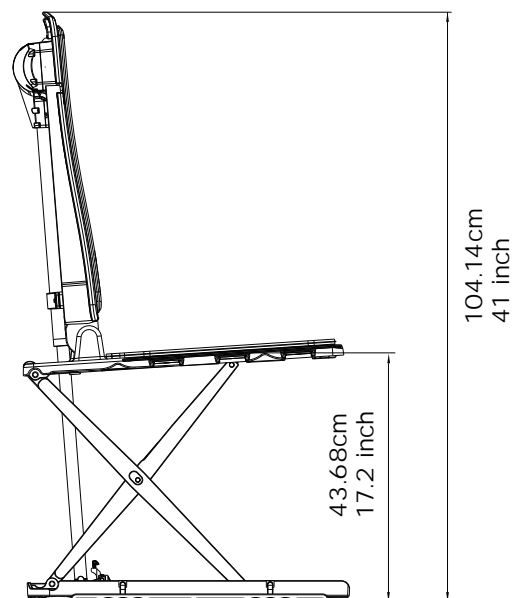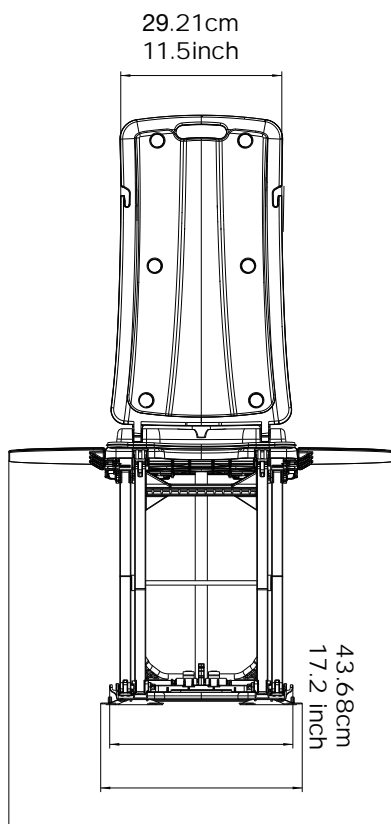Operating temperature	5-40°C
Operating humidity	Under 40°C 20%-90%
Operating air pressure	700-1060 hPa
Operating altitude	≤2000m
Storage temperature	-10-50°C
Storage humidity	20%-90%
Storage air pressure	700-1060 hPa
Charging voltage	100-240V 50/60Hz max0.8A
Battery pack	14.8V 2.2Ah
Specified load	1400N
Lowest point height	60mm±5mm
Highest point height	480mm±5mm
IP grade	IP68

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