

PLEASE READ INSTRUCTIONS BEFORE USE

INSTRUCTION MANUAL

Customer Service:


If there is anything wrong with our product, please kindly contact us without any hesitation before you use it. We will spare no effort to help you solve the any problem. Each email will be replied to within 24 hours, excluding weekends and Holidays.

Warranty Information:

"Defective Upon Arrival" claim should be made within no more than 30 days after delivery, so please kindly check the condition ASAP once delivered. We provide a 1 Year Warranty for replacement parts (excludes accidental damage). If there is any other thing we can do for you, please feel free to contact us.

Important Notice:

Please read the following instructions carefully before using. Assure all screws are completely through the holes and legs are all the same height. Periodically check to assure that all screws remain tight.

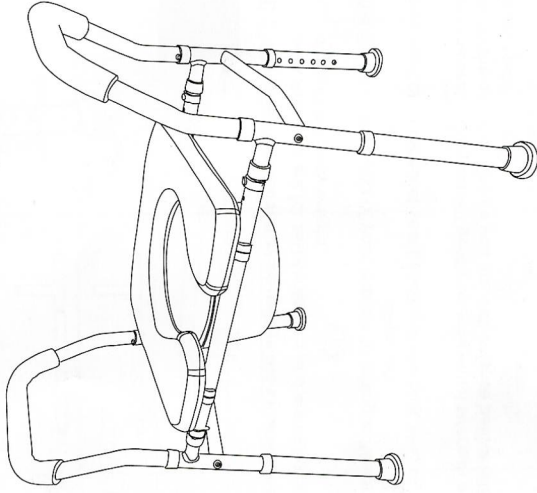


Raised Toilet Seats

WARNINGS

- DO NOT install commode without first reading and understanding this assembly and instruction sheet. If you cannot understand these instructions, contact us directly before attempting to install. Otherwise, injury or damage may occur.
- DO NOT exceed the maximum user weight capacity of 330 lbs.
- Inspect all parts for shipping damage before assembly. If there is any shipping damage, DO NOT USE, and contact us for further instructions.
- Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- All four leg extensions with rubber tips must touch the floor simultaneously at all times.
- Users with limited physical strength should be supervised or assisted while using commode.

PARTS LIST


All of our products have been carefully inspected in the factory and confirmed to be free of defects and complete. After delivery, please carefully check over the product for possible shipping damage.

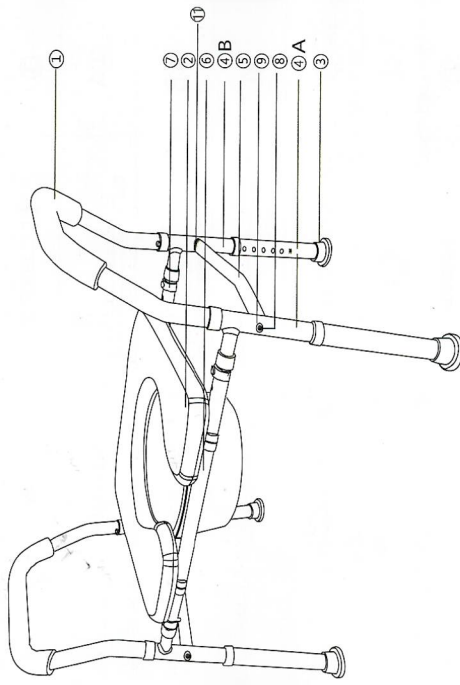

STEPS FOR ASSEMBLY


1

- Please connect the tubes marked with the same characters on Part ① Part ④, then connect the Part ⑥.

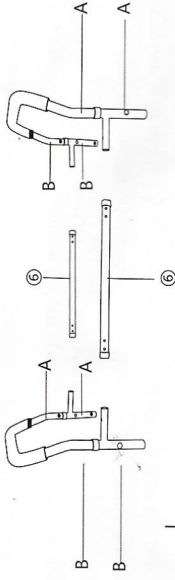

2

- Please align the holes and insert the c-clips (Part ⑦).

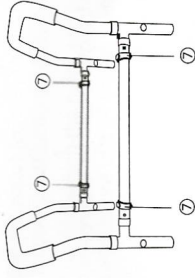

3

- Please connect the tubes (Part ⑤) with Side Frames (Part ④), and then fixed them by Screws (Part ⑧).


3

Part Number	Part Image	Description	Quantity
①		Armrest	2
②		Seat base with Splash Guard	1
③		Leg	4
④A		Side Frame	2
④B		Side Frame	2
⑤		Side Crossbar	2
⑥		Horizontal Crossbar	2
⑦		C-Clip	4
⑧		Screw	4
⑨		Washers	4
⑩		Screwdriver	1
⑪		Side Crossbar Washer	4

2

ADJUSTMENT OF HEIGHT


1. Before use, adjust the height of each leg to your specific needs make sure all legs are at the same height. When the ideal height is reached lock legs into place by aligning buttons into proper holes per illustration.
2. To be sure that legs are locked into place by pushing or pulling legs.

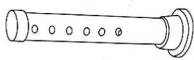

TIPS


⚠ NOTE : The raised toilet seat may be used in the same manner as a regular seat. Armrests are for assistance only and are not designed to support full body weight.

1. Back up to the seat until the backs of your legs are touching it. Reach back for the armrests.
2. Gently lower yourself onto the toilet seat bending at the hips, elbows, and knees as needed.
3. To get up from the seat, push yourself gently up with equal strength exerted from both arms on the armrests. Ensure you are well balance before releasing your grasp from the armrests.

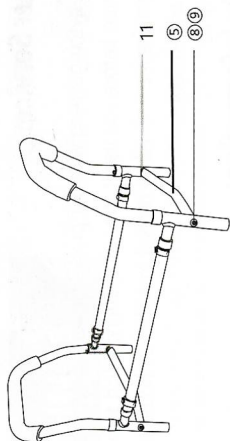


4

- Please install the seat base(Part 2) to the frame(Part 6).

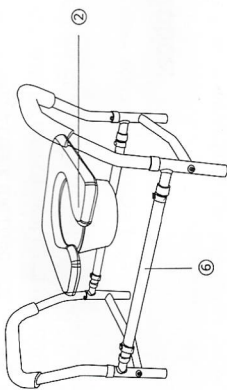

5

- Insert the legs(Part 3) to the side frame(Part 4).

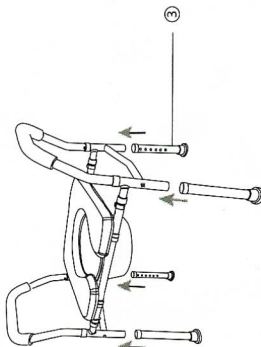

6

5