

Carousel Sliding Shower Chair

Instructions for assembly and use

Step 1. Open the master carton, take out parts from the plastic bags in the carton.



Step 2. Insert the two bent legs into the frame of the shower chair until the adjustment copper buttons pop out of holes. Please kindly note that only two of them will show up. The other two need to be pressed so that the legs can slide into the tubes.




Step 3. Assemble the four legs on to the chair frame leg hole.

Step 4. Insert the backrest into the frame of the sliding chair until the buttons being engaged in the hole.

Step 5. Adjust the legs to desired heights. Please kindly note that the legs can be adjusted 1/2 inch at one time by pushing the button.

Steps Of Operation


Step 1. Press the middle gray button to engage to make sure the sliding chair can slide. Slide the chair to the left or right until it gets to your desired place. Press the red button and it would rotate. For every 90 degrees, Chair would be locked into a regular place.

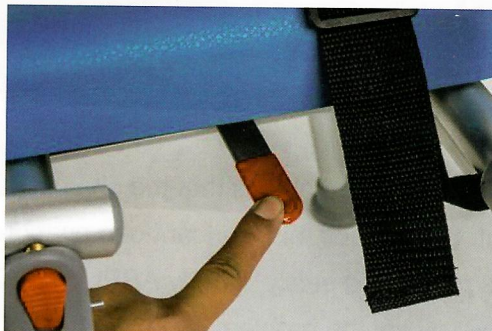

Step 2. Prior to showering, push the red lever under seat and turn the chair to make it face the user or the nursing staff. Once the user is seated and strapped into the chair, push the red lever and have the user rotate into the tub one foot at one time. Once done, push the gray button under the seat to slide the seat into the tube and rotate to a desired place. Then locks it.



Step 3. The chair has flip up locking arms for easy transfer. When needing leaving the tub, push the gray button to engage the sliding chair. Slide the chair until it's locked. The red lever can be pressed to rotate the chair. Unlatch the seat belt and exit the chair.

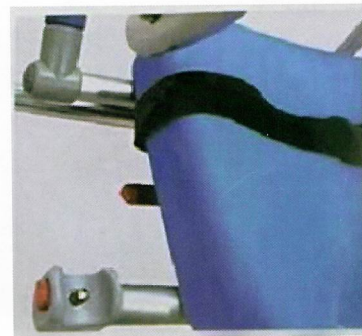

Note:

1. The carousel sliding shower chair can also be used in a shower as shown below.
2. The functionality is the same whether used in a shower or bathtub.



Not sliding easily? --How to fix?

3. Please note that a light lubricant is applied to horizontal rails that the chair slides on. This lubricant is applied at the factory before the item is shipped out. Sometimes this lubricant can evaporate if high temperature occurs during the shipping process. If your chair is not sliding easily, simply APPLY A LIGHT COATING OF WD40 LUBRICANT to both horizontal rails. This will rectify the problem.

Shower chair is not level in the shower or bathtub? ----How to fix?

Rotate the feet to accommodate the contour or shape of the tub or shower to make sure they are flat to the surface. Make sure the buttons on leg adjustments are positioned correctly so the chair is completely level.

IMPORTANT SAFETY INFORMATION

READ ALL SAFETY INFORMATION BEFORE USING

It is the user's responsibility to ensure the appropriateness of this device for the user in question before using.

Never use this device with individuals who are severely mentally, cognitively, physically or vision compromised.

Always attach safety belt and adjust to a snug fit around user's hips.

Never leave a compromised user unattended.

Always ensure that the legs are properly adjusted to ensure that the bench is level side to side and front to back.

Never exceed the maximum weight capacity of 330lbs.

Note: Failure to follow the above noted safety instructions can lead to severe injuries.